

HOW TO FIT FILTERS TO ANTIBACTERIAL MASK




STEP 1 - Locate page of double sided tape. (There will be one strip for each side per filter)



STEP 2 - Carefully peel back tape with backing (be careful, as the tape can stay stuck to sheet). It is best to fold it back on itself to get the tape to peel with backing



STEP 3 - Apply tape vertically to line up with crosses on filter.

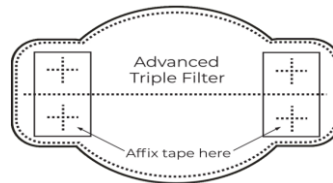


STEP 4 - fold filter in half to match contour of mask, then peel one side off and stick down to mask and press together firmly. Then repeat with the other side. Done!



Please replace filter daily

About the Filter:

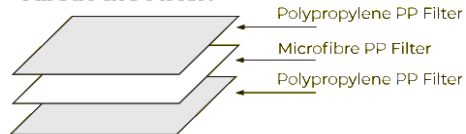

FRESH AIR TRIPLE LAYER DISPOSABLE MASK FILTERS

10 Pack

10 Pack - Advanced Triple Filter
Change filter regularly

10 PACK FILTERS
Part No. MMS921

Made for My Medical Supply™ in Korea
www.mymedicalsupply.com.au

0655887820519

Like us on Facebook

Filter fits all My Medical Supply™ Aerosilver / Urethane and True Colours team face masks.

How to use:
1. Use enclosed double-sided tape to fix filter to mask.
2. Replace filter daily.

About the Filter:

Product Name: Fresh Air Filter
Product Size: 140mm x 90mm
Material: Polypropylene + Microfibre
Pouch Size: 250mm x 140mm
10 sheets per pack